

SEKISUI LATEX AGGLUTINATION TEST SLIDE

Sekisui Latex Agglutination Test Slide is a disposable latex agglutination slide developed by the advanced polymer technology of Sekisui Chemical Co., Ltd.

It has equivalent surface nature to a glass slide and affords an efficient agglutination assay work and judgement.



1 Advantages

1) Hydrophilicity of the surface is similar to a glass slide so that the reagent and the specimen can be readily mixed, and lead to a reliable, reproducible agglutination reaction. As a result, the assay operation is very efficient.

2) The disposable nature eliminates troublesome cleaning of slides and undesirable effect of contaminated slide on assay results. Also it reduces the risk of infection to operators caused by accidental contact to specimen


3) Write-on space is provided for each ring for marking with pencil or ball-point pen.

4) The ring is marked with a hydrophobic ink to assure good encircling and prevents the liquid from flowing out while mixing reagent and specimen. The ring will also retain the test mix within it during the shaking by an electric rotor.




2 Specification of product

1) Standard product : 3 rings or 6 rings in red, on a black or white slide

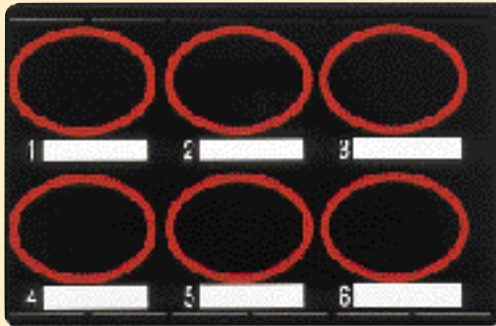


6 rings on black

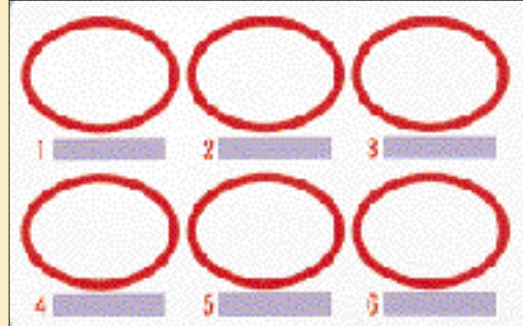

6 rings on white

2) Custom-made product :

Various sizes, designs and packaging are available on your request.

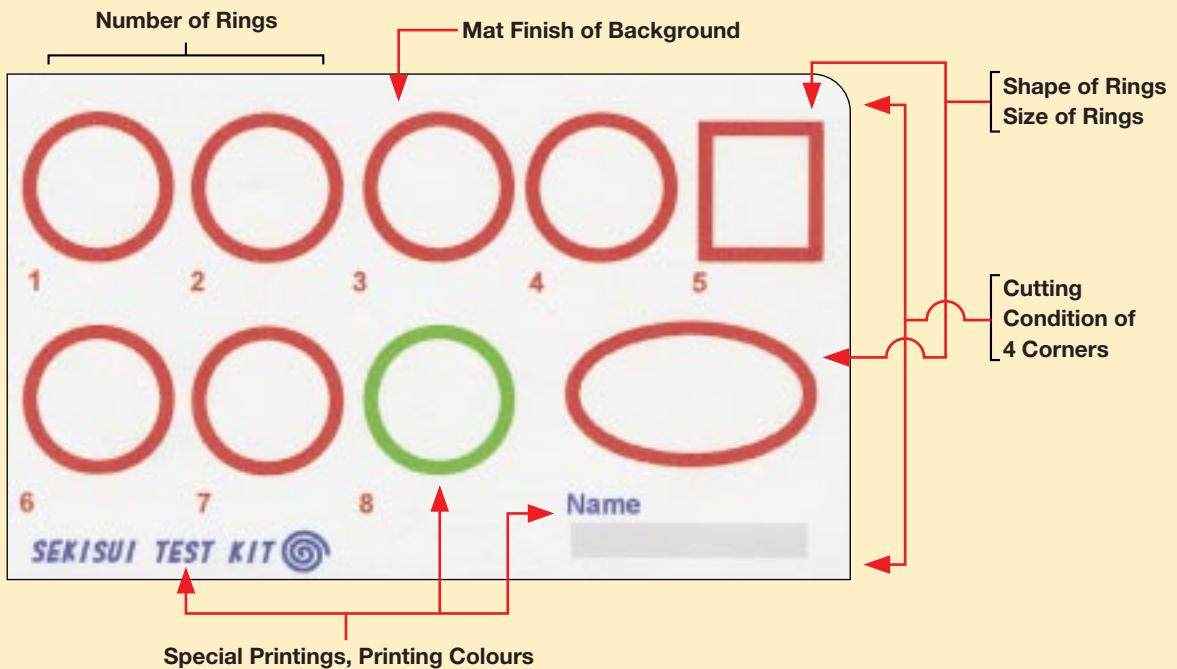

3 Precaution

- 1) Do not touch the surface of the test slide, especially the surface in the ring.
- 2) Do not wipe the surface of the test slide with gauze, cotton wool or use any solvents including water and alcohol, as they may reduce the hydrophilicity of the surface and give undesirable effects on the agglutination reaction.
- 3) The test slide is disposable and for 'one use' only. Do not use it repeatedly.
- 4) Store at room temperature in a clean atmosphere avoiding high humidity, high temperature and keep out of direct contact with the sun. For longer-term storage the recommended temperature is around 10 °C.

SEKISUI CHEMICAL CO., LTD.

Pharmaceutical & Diagnostic Division
High Performance Plastics Headquarter
2-4-4, Nishitenma Kita-ku, Osaka, 530-8565
Japan

Tel: +81-6-6365-4157 Fax: +81-6-6365-4162

Web: www.sekisui.co.jp/medical/index_e.html

The color tone may sometimes differ from the original due to printing effect.

SEKISUI CHEMICAL GmbH

Immermannstrasse 65C, 40210 Dusseldorf
Germany

Tel: +49 211 36977-0 Fax: +49 211 36977-31